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Adjusting the handbrakes

How to adjust the handbrakes

The following information is designed to assist you with the adjustment of your NOVA Feather Touch Handbrake System. **Note:** The braking system on your new NOVA walker has been preset from the factory.

To maintain this preset position, be sure that the tension adjustment screw locking nuts are tight against the housing and the rear wheels (**image 14**).

If you are still having difficulty with your brakes, you can adjust them by doing the following:

1. Handbrake is too loose

Loosen the tension adjustment screw locking nut and turn the screw one full turn clockwise (as viewed from standing at the back of the walker). If this does not correct the adjustment, repeat one full turn at a time until you have adjusted your brakes. Make sure the locking nut is spun tight against the housing to prevent the screw from loosening.

2. Handbrake is too tight

Loosen the tension adjust screw locking nut and turn the screw one full turn counter-clockwise (as viewed from standing at the back of the walker). If this does not correct the adjustment, repeat one full turn at a time until you have adjusted your brakes. Make sure the locking nut is spun tight against the housing to prevent the screw from loosening.

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Warranty

Warning

Walker must be in a stationary position, on a level surface with the brakes locked when sitting or attempting to sit. **DO NOT** scoot around or navigate the walker when in the seated position. Risk of serious damage to the walker and serious injury or death may result if brakes are not in the locked position.

Maintenance

Check your walker periodically to make sure that the brakes are working correctly and that all of the nuts and bolts are secure. **DO NOT use the walker if the brakes are not working correctly.**

Warranty Information

NOVA stands firm on our commitment to providing the highest quality products, as well as exceptional service. This product was built to precise standards and thoroughly inspected prior to shipment. This

warranty represents our confidence in the materials and workmanship of our product. NOVA extends this warranty only to the original purchaser of this product. The warranty does not extend to any subsequent purchaser or owner. The warranty is void upon any sale or transfer of ownership or use by other person. NOVA warrants its products are free from defects in material and workmanship under NORMAL use for the period commencing upon the date of purchase and continuing for the following specified period of time after that date:

Metal frame:	Limited Lifetime Warranty
Brakes:	5 Years
Non-Metal/Metal parts:	1 Year Limited Warranty (Wheels, Hand Grips, Padding, Cables, Upholstery, Bag, Screws, Plastic Parts)

This warranty does not cover problems due to user negligence, misuse of the product, failure to adhere to product instructions, or normal wear and tear. Items that have been subject to negligent abuse, modified without prior written authorization are not covered within this warranty. For warranty service, please contact the dealer from whom you purchased your NOVA product. In the event that you do not receive satisfactory service, please call us directly at the number below. Be prepared to indicate, the nature of the defect, the product serial number, and the name and location of where you purchased the product. You will also be required to provide a receipt of your purchase. DO NOT return any products to our office without prior consent. Repair or replacement is the only remedy under this limited warranty. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of product.

THIS WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES, IMPLIED WARRANTIES, IF ANY, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. NOVA SHALL NOT BE LIABLE FOR ANY SEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.

*****Remember to keep the product's serial number and the date of purchase*****

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